



InVoCell®

Discover an innovative option
used in treatment that considers
your concerns about cost.

Clinic name:

Website:

Phone number:

CONCEIVE
HOPE
WITH INVOCCELL

LEARN MORE AT **INVOCCELL.COM**



© 2019 Ferring B.V. 04/19 INVO/329/2019/USa

INVOCCELL is a registered trademark of INVO Bioscience. FERRING, the FERRING PHARMACEUTICALS logo, and other Ferring product trade names are registered trademarks of Ferring B.V.

Discover INVOCELL

An innovative option used in fertility treatment that considers your concerns about cost.

INVOCELL was designed to offer you another option and hope on your fertility journey. INVOCELL is an intravaginal culture (IVC) system and consists of the INVOCELL culture and retention devices. The **first and only** FDA-cleared device of its kind used in fertility treatment, INVOCELL uses your body as an incubator for fertilization, making you an even greater part of the process.

INDICATIONS FOR USE

INVOCELL® Intravaginal Culture System contains the INVOCELL Culture Device and INVOCELL Retention Device. The Culture Device is indicated for use in preparing, holding, and transferring human gametes or embryos during In Vitro Fertilization/Intravaginal Culture (IVF/IVC) and Intra-cytoplasmic Sperm Injection Fertilization/Intravaginal Culture (ICSI/IVC). The Retention Device is used during the incubation period to aid in holding the Culture Device in the vagina. The Culture and Retention Devices are not indicated for periods of use exceeding 72 hours.

SELECT IMPORTANT SAFETY INFORMATION

- Do not use the Culture or Retention devices if you are unable to tolerate having a device placed within your vagina.


Please see continued Select Important Safety Information on the following page and accompanying Patient Information.



In a clinical trial, about 1 in 4 embryo transfers resulted in a birth


INVOCELL was studied in a clinical trial involving 450 cycles, 421 of which resulted in embryo transfers. Of these transfers, **about 1 in 4 resulted in a birth.**

It's important to know: adverse pregnancy events, like miscarriage and pre-term births, were reported in the clinical study as these were primarily related to the effectiveness of the device (live birth rate).

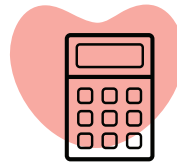

An innovative option that considers your concerns about treatment costs

The use of mild ovarian stimulation and fewer monitoring visits, as decided by your doctor, could reduce some of the costs of treatment with INVOCELL. Talk to your doctor about what is best for you.

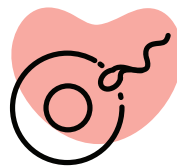

Unique method that uses your own body as an incubator

Your doctor will place the eggs and sperm in the INVOCELL device, and then position the device in your vagina. Your body provides the environment needed for fertilization and incubation to occur.



Used with mild ovarian stimulation

Before INVOCELL is placed in your body, your doctor will prescribe medicine to help your body produce eggs. He or she will then retrieve the eggs from your ovaries and place them in the INVOCELL culture device with sperm, to let them fertilize and incubate in your body.



You may require fewer monitoring visits, as decided by your doctor

With INVOCELL, you may need fewer monitoring visits. Your doctor will work with you to find out the right number of follow-up visits for your needs.

INVOCELL, IVC, & You

An easy way to think about it is that INVOCELL is the device used in the IVC procedure. During IVC, your doctor will place eggs with sperm in the INVOCELL culture device. Then, he or she will position the device in your vagina for fertilization and incubation to occur.

A CLOSER LOOK AT INVOCELL

INVOCELL has two parts:
the culture device and the retention device.



SELECT IMPORTANT SAFETY INFORMATION

- Do not use the Culture Device if you are sensitive to medical grade silicone or polystyrene or if you have a severe case of vaginitis or a history of toxic shock syndrome. Tell your doctor if you have had any recent pelvic surgery.
- Avoid any activity such as sexual intercourse, strenuous physical activity, swimming, bathing in a tub (a shower is permissible), use of a douche, sauna, or any other activity that may alter the temperature of the vaginal cavity and avoid handling or removing the Culture and Retention Devices while in place.


Please see continued Select Important Safety Information on the following page and accompanying Patient Information.

HOW IVC WORKS WITH INVOCELL

First, your doctor will prescribe medicine so that your body can produce eggs. Then, your doctor will retrieve the eggs from your ovaries and place them with sperm in the INVOCELL device.



1 The eggs and sperm are placed in the INVOCELL culture device.

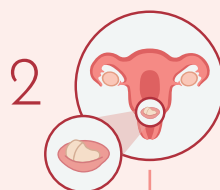

2 Your doctor will then place the culture device inside the retention device, and position them inside your vagina.

YOU SHOULD KNOW: There shouldn't be any discomfort while the device is in your body. If there is, call your doctor right away.

AND REMEMBER: During this time, avoid any strenuous physical activity or any activity that could change the temperature of your vagina. So—no sexual intercourse, no taking a bath, no saunas, and no douching. Ask your doctor if you have any questions about what you can and can't do while INVOCELL is in your body and read the Patient Information.



3 After this incubation period, your doctor will remove the device from your body and examine the embryos that have developed.



4 Then, your doctor will work with you to decide on the number of embryos to be transferred to your uterus. And the good news is, you can freeze any remaining embryos.

Ready to take the next step?

Being prepared with the right questions—like the ones below—is a good place to start.

- Is INVOCELL right for me and my fertility challenges?
- How does IVC with INVOCELL work?
- How is IVC with INVOCELL different from other options used in fertility treatment?
- What are my chances of getting pregnant with INVOCELL?
- How long does IVC with INVOCELL take?
- How many monitoring visits will I need?
- How much does an IVC cycle with INVOCELL cost? Does the cost include medication?
- Will my insurance cover treatment with INVOCELL?

SELECT IMPORTANT SAFETY INFORMATION

- Your doctor should explain to you what to do if the Culture and Retention Device should shift in the vagina or fall out. Call your doctor right away should this occur.

Your doctor will determine if this product is right for you and will provide you with more detailed information about the risks and benefits associated with using this device. Always follow your doctor's instructions. Contact your doctor immediately if you have any questions or concerns before, during, or after treatment.

Please see Select Important Safety Information throughout and accompanying Patient Information.

Remember: your fertility specialist is your best resource as you navigate your fertility journey. So, make sure to ask your questions, voice your concerns, and discuss what's right for you.

Ask your fertility specialist about
INVOCELL

